



Cygnetic by Swann-Morton

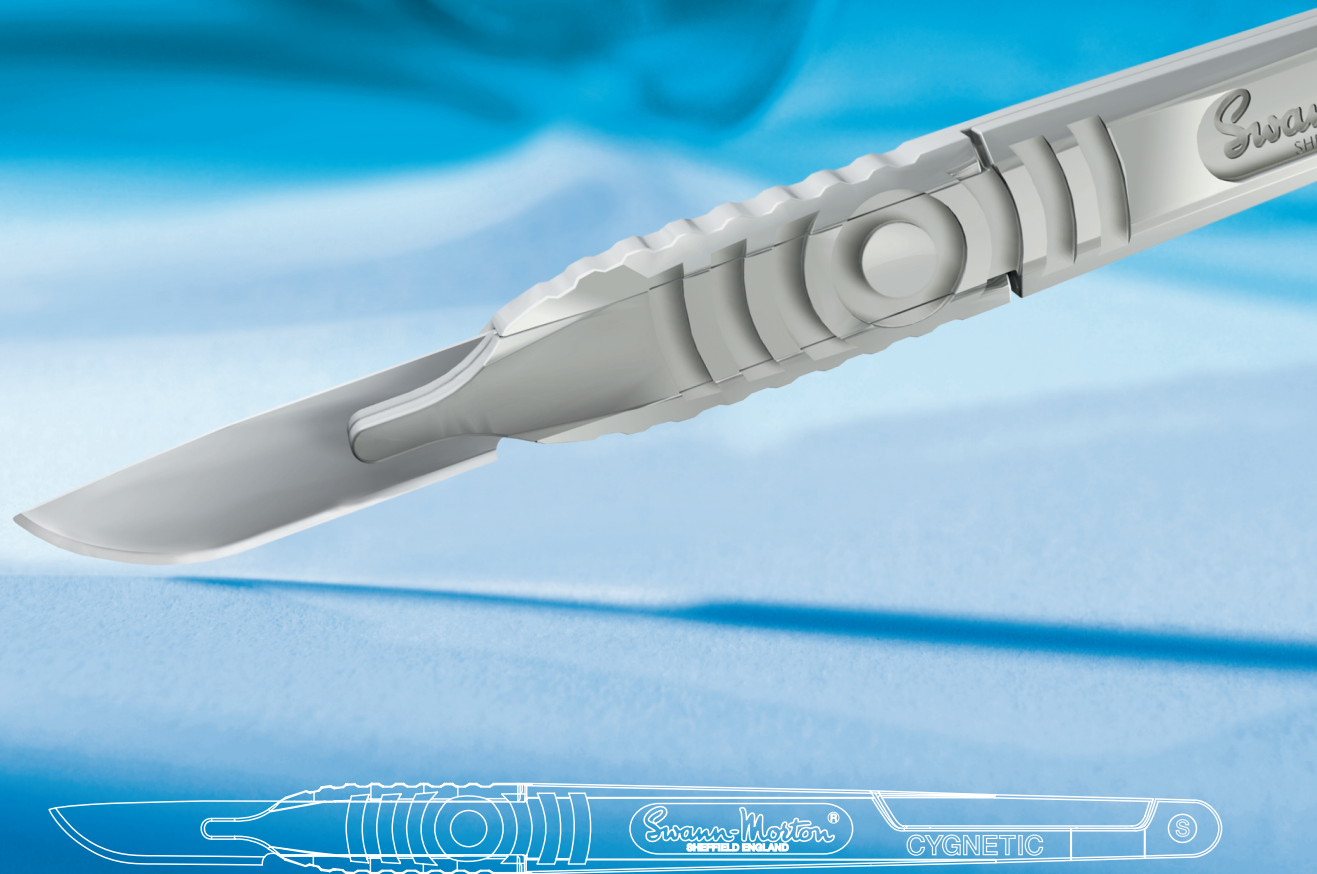

For use within rigorous Orthopaedic and General surgical procedures.

"Cygnetic" provides the surgeon with additional strength and stability through the patented combination of its specially designed blade and handle.

Its functionality will assist the nurse with safe and easy blade attachment and removal whilst the simple disassembly for cleaning will be welcomed by the decontamination department.

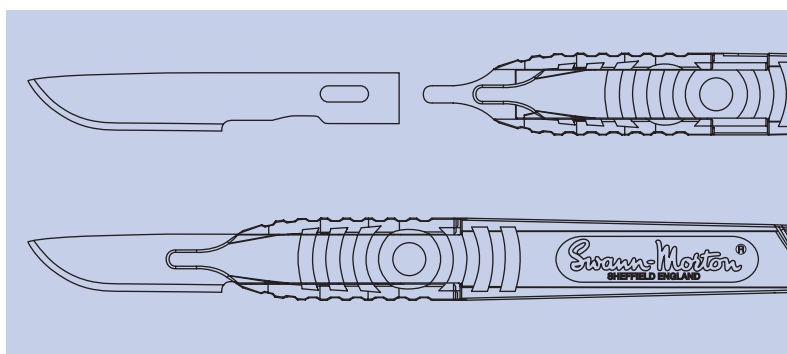

Easy to attach, easy to detach and easy to clean.

Swann-Morton®
Sheffield - England



CE
All medical devices are CE marked in
accordance with the Medical Device
Directive (93/42/EEC)



Swann-Morton®

THE WORLD'S LEADING SURGICAL BLADE MANUFACTURER


Cygnetic by Swann-Morton

For use within rigorous Orthopaedic and General surgical procedures.


Easy to attach, easy to detach and easy to clean

Easy to attach

To load the blade, lift the lever to the vertical, the main body slides back to accommodate the blade.

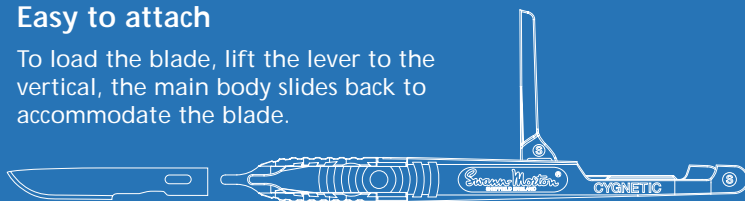


With the blade in place, push the lever back, the main body slides back to lock the blade in position.

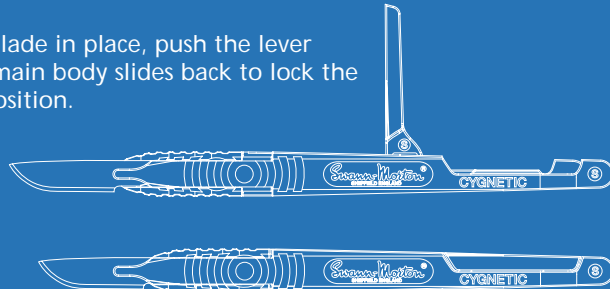


Easy to detach

Simply reverse the previous operation and dispose of the used blade safely.
No need to handle the contaminated blade.

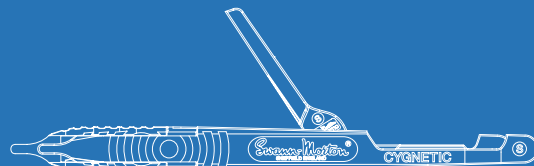


Easy to clean

With the blade removed, push the lever forwards until it 'stops' in the forward position.

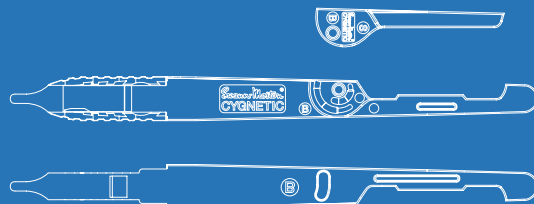

The body will separate into two pieces and release the lever for cleaning in the recommended manner.

Refer to our website at:
www.swann-morton.com for cleaning and reprocessing instructions.

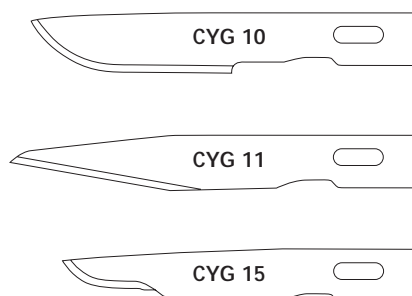

All three blades sizes (10, 11 and 15) are available in boxes of 50, Sterile, individually V.P.I. protected in Metal Foil Packets.

Product	Product No	Unit
---------	------------	------

Cygnetic Range Blades - Sterile

Blades in Stainless Steel - Sterile, individually V.P.I. protected in Metal Foil Packets

10	5301	Box of 50
11	5303	Box of 50
15	5305	Box of 50

Cygnetic Range Handles - Non-Sterile

Handles in Stainless Steel - Non-Sterile, individually sleeved

Cygnetic	5398	1
----------	------	---

Swann-Morton and the 'ring pattern' logo are registered trademarks of Swann-Morton Limited and related companies. The products shown are manufactured for use by trained health care professionals and can cause injury if used or handled incorrectly. The information and illustrations used in connection with the products shown on this leaflet are for general guidance only and users must seek proper training and instruction in their use before using or handling such products. Swann-Morton Limited exclude all liability for injury to the extent permitted by law.

Swann-Morton®
Sheffield - England

Swann-Morton Ltd. Penn Works, Owlerton Green, Sheffield S6 2BJ
Telephone: 0114 2344231, Sales 0114 2344223, Fax: 0114 2314966
uksales@swann-morton.com, exportsales@swann-morton.com
orders@swann-morton.com, www.swann-morton.com

CYG/issue 1